

# INSTRUCTIONS FOR

## Reusable Full Face Shield (#20413)

### Items included in kit:



1 x 3D printed halo  
(Body contact component)  
Material: Polyamide 12 or Digital  
Production Resin 10 (Black or White)



5 x Face shield  
Materials: Polycarbonate/Polyethylene  
Terephthalate Glycol-modified



1 x Tightening strap  
(Body contact component)  
Material: Synthetic polyisoprene (non-  
latex)

### Preparation

Remove protective, opaque coating from both sides of the Face Shield until it is fully transparent.

### Insert shield into halo

Place the Halo on a flat surface and orient as shown with notches facing up.



Insert one corner of the shield into the halo slit.

Continue to insert the shield fully into the halo slit with a gentle rocking motion while applying downward force.



### Insert tightening strap into halo

Insert the non-buckle end of the tightening strap through the loop in the halo.

Note: the smooth side of the strap should be facing towards the halo. Take the metal buckle on the tightening strap and fit it over the hook on the halo. Take the hook portion of the tightening strap and fold it over onto the loop portion. Place the completed face shield on your head and tighten the strap to comfort.



### Cleaning information

Follow your facility's internal procedures for disinfection. The following cleaning materials may be appropriate:

- Soap-water
- Isopropanol (70%) wipe

The following are not compatible with the materials and should not be used to clean the face shields:

- Hydrogen peroxide vapor sterilization
- Autoclave

*Caution: Non-surgical device with minimal-to-low barrier protection. Not for use during surgical or invasive procedures.*

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923  
Web: GoHCL.com • Email: hcl@GoHCL.com