




# INSTRUCTIONS FOR Mobile Hygiene Station & Accessories (#18384-18395)

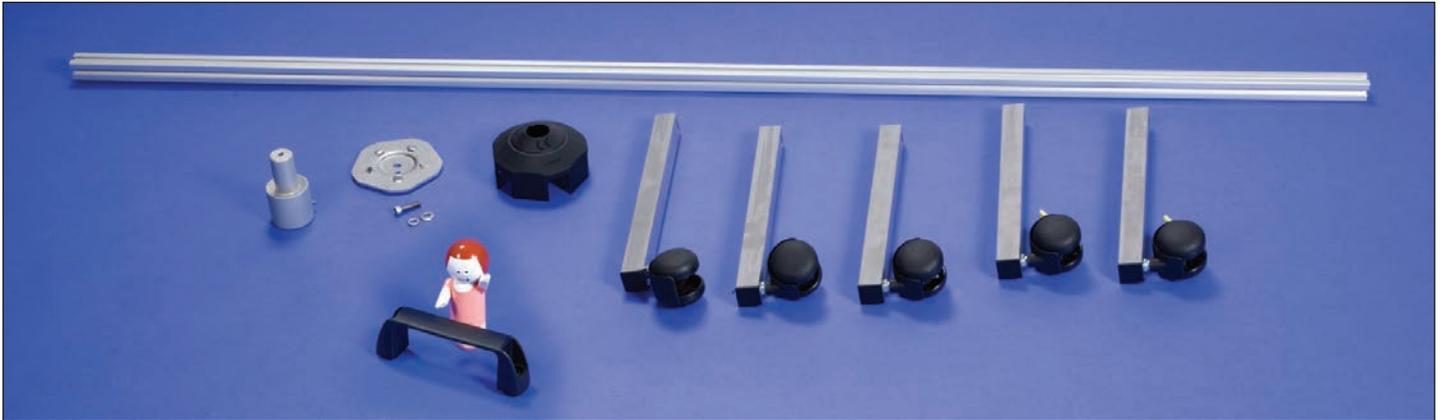


**Step 1.** Place the pole adapter into the black plastic housing.

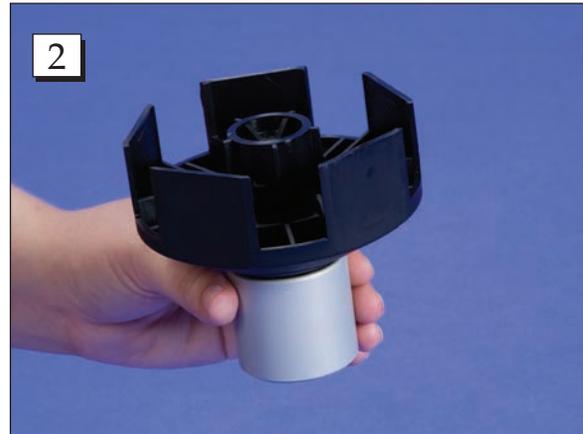

**Step 2.** Flip the unit upside down per the illustration above.

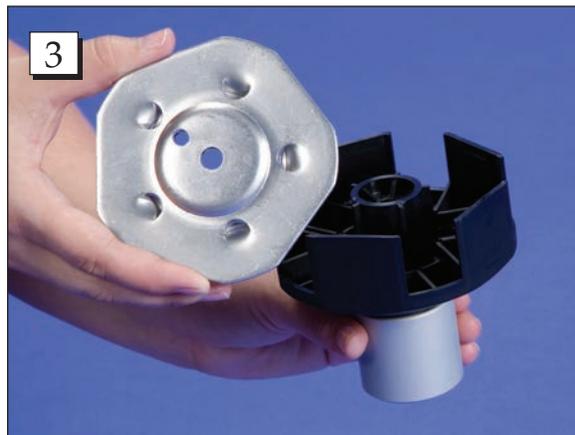


**Step 3.** Place the cover plate on the black housing.

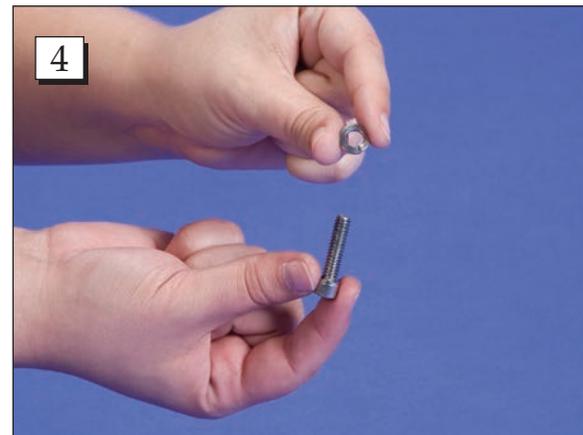

**Step 4.** Place a lock washer first followed by a flat washer over the screw.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923  
Web: GoHCL.com • Email: hcl@GoHCL.com

# INSTRUCTIONS FOR Mobile Hygiene Station & Accessories (#18384-18395)

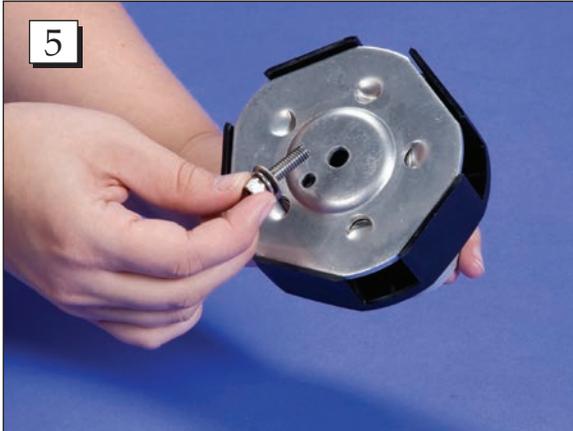

**Step 5.** Insert the screw through the hole on the cover plate into the pole adapter and thread the screw in three turns.



**Step 6.** Tilt the cover plate up and insert a leg section with the open hole in the leg facing upward toward the cover plate.



**Step 7.** Repeat step six on the remaining leg sections.



**Step 8.** Use a  $\frac{5}{32}$  allen wrench to tighten the screw.




**Step 9.** Lay the base assembly upright on the floor.



**Step 10.** Back the set screw out of the pole adapter.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923  
Web: GoHCL.com • Email: hcl@GoHCL.com

# INSTRUCTIONS FOR Mobile Hygiene Station & Accessories (#18384-18395)

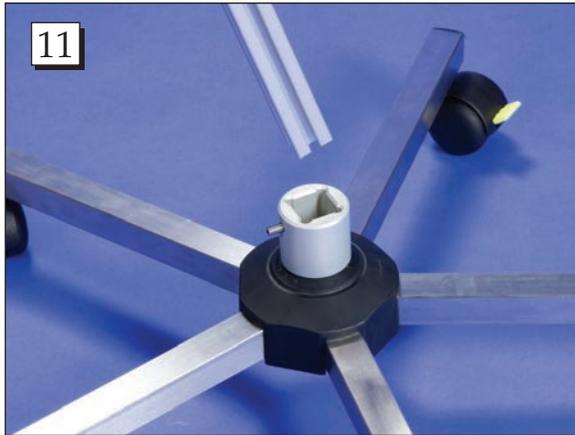

**Step 11.** Place the pole section into the pole adapter.



**Step 12.** Tighten the set screw snug against the pole section.



**Step 13.** Slide the T-Nuts on the handle onto the pole.



**Step 14.** Tighten the screws in the desired location.



**Step 15.**

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923  
Web: GoHCL.com • Email: hcl@GoHCL.com

# INSTRUCTIONS FOR Mobile Hygiene Station & Accessories (#18384-18395)



**Step 16.** Slide the T-Nuts on the organizer onto the pole.



**Step 17.** Tighten the screws in the desired location.

**Note:** Use the same process to attach all of the accessories, which are sold separately.



**Call Free 1.800.848.1633 • Fax Free 1.800.447.2923**  
Web: [GoHCL.com](http://GoHCL.com) • Email: [hcl@GoHCL.com](mailto:hcl@GoHCL.com)