

INSTRUCTIONS FOR Portable IV Pole, 2-Hook (#1575-01)

PRECAUTIONS

- Never use for IV bags of more than 1200 ml (or 2.5 lbs).
- Pole is most stable at lower height.
- Patient should remain stationary during use.
- Never pull on IV tubing as this may cause pole to tip over.
- Always use attached Velcro to secure IV tubing to pole.
- Never lean on pole or use for support.
- If using rate-controlled gravity set, follow the manufacturer's instructions.

HOW IT WORKS

1. Remove the IV pole from the box.
2. Determine if pole will be placed on a table top or floor.
3. Pull legs outward from the center of the pole and lock into place to secure the base (Figure 1).
4. Raise pole sections to desired height and twist each silver pole section firmly clockwise to lock into place (Figure 2).
5. Hang IV bag on the hook (Figure 3).
6. Secure tubing to pole with Velcro at level of IV site.
7. To loosen and collapse pole, first remove IV bag and tubing, then twist each pole section counter-clockwise.
8. When no longer needed, place pole in regular recycling.



Figure 1



Figure 2



Figure 3



CLEANING INSTRUCTIONS FOR Portable IV Pole, 2-Hook (#1575-01)

Stainless Steel IV Stands must be cleaned on a regular basis in order to prevent unnecessary damage to the stainless steel surface. When cleaning the IV Stands, make sure to use the proper cleaning agents and cleaning materials.

CAUTION: DO NOT USE the following materials on the stainless steel surfaces of the IV Stands:

Abrasive Pads
Scrapers
Steel Wool
Wire Brushes

CAUTION: DO NOT USE the following agents on the stainless steel surfaces of IV Stands:

Hard water
Bleach or any products containing Chlorine
Hydrochloric Acid
Denatured Alcohol

NOTE: Always follow the manufacturer's directions for any chemicals used on the stainless steel surfaces of the IV Stands.

Use the following cleaning materials and agents:

Soft clean lint-free cloth
Distilled water
Distilled water with mild detergent
Cleaners approved for use on stainless steel

How to clean a Stainless Steel IV Stand

1. Using a damp, lint-free cloth and approved cleaner, wipe down all stainless steel surfaces of the IV Stand.
2. Use a lint-free dry cloth to dry the IV Stand, or let air dry. If air drying, ensure IV Stand is completely dry before storing. Store in a cool, dry place.

Call Free: 1.800.848.1633 • Fax Free: 1.800.447.2923

GoHCL.com • hcl@GoHCL.com

© Health Care Logistics, Inc. 2024