


INSTRUCTIONS FOR

Easy Fill Vial Plug Dispenser (#7339)

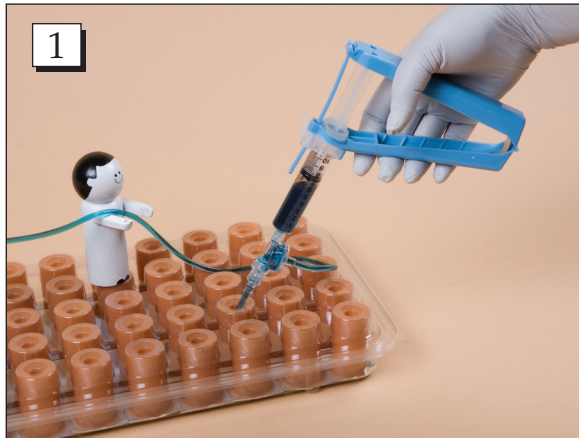

Step 1. Using the holder tray, fill the vial through the hole in the bottom using a Dispensing Syringe, HCL #3221 or automatic filling pump. We recommend starting at one end of the row of vials and working systematically down the line, row by row.



Step 2. Plug the hole using the Easy Fill Vial Plug Dispenser, HCL #7339. Fill the dispenser with a bag of plugs.



Step 3. Position the dispenser over the vial and push down on the plunger.



Step 4. Label your vials as needed. Then vials are ready to be distributed to your patients.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923
Web: GoHCL.com • Email: hcl@GoHCL.com