

INSTRUCTIONS FOR Lock-To-Pole IV Lock Box with Key Lock (#17938)




Step 1. Inspect the clamps for signs of wear and tear. Open the door of the IV box and make sure the two clamps are open. Note: The box will not properly attach to the pole if the clamps are not in the open position.



Step 2. Align the clamps on the inside of the box with your pole. Holding the box with your four fingers and the pole with your thumb, press firmly until the clamps wrap around the pole. Once installed, do not twist the box on the IV pole.



Step 3. Hang the IV bag on the hook.

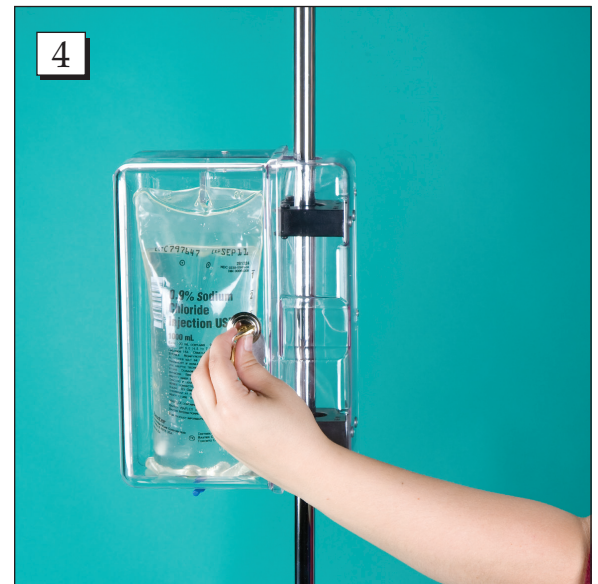

Step 4. Lock the IV box door.

Health Care Logistics will not be liable if product is used outside of scope of the instructions.

Call Free: 1.800.848.1633 • Fax Free: 1.800.447.2923
GoHCL.com • hcl@GoHCL.com

© Health Care Logistics®, Inc. 2024

CLEANING GUIDE FOR Lock-To-Pole IV Lock Box with Key Lock (#17938)

Wipe surfaces with 70% Isopropyl Alcohol, taking care to avoid electronic components. Allow to air dry per dwell time provided on disinfectant label.

Warnings:

Disinfectant should be compatible with materials listed below. Some disinfectants are not compatible with polycarbonate. Discontinue use if signs of damage occur.

Materials:

Polycarbonate
POM (acetal)
TPE (Thermoplastic Elastomer)
Polypropylene

Alternate Disinfectant Option:

19954, PREempt™ One-Step Surface Cleaner and Disinfectant Wipes, Canister
19954-31, PREempt™ One-Step Surface Cleaner and Disinfectant Wipes, Case