

INSTRUCTIONS FOR

“The Crusher” Stainless Steel Tablet Crusher (#7349-01)



Step 1. Place tablet(s) into a ½ oz. crush cup.



Step 2. Cover with another crush cup and place on tablet crusher.



Step 3. Use an up and down tapping motion to crush the tablet.



Step 4. Remove the cups from the crusher and administer.

Call Free: 1.800.848.1633 • Fax Free: 1.800.447.2923
Web: GoHCL.com • Email: hcl@GoHCL.com