

# ASSEMBLY INSTRUCTIONS FOR Topi-Click (HCL# 17562, 17894, 17895)



Step 1. Remove the cap.



Step 2. Fill Topi-Click with desired topical ointment up to 33mL.




Step 3. While filling, stop at each  $\frac{1}{3}$  fill to tap the Topi-Click to help settle the topical ointment.



Step 4. Replace cap and press till the cap clicks on securely.



Step 5. Replace outer cap.

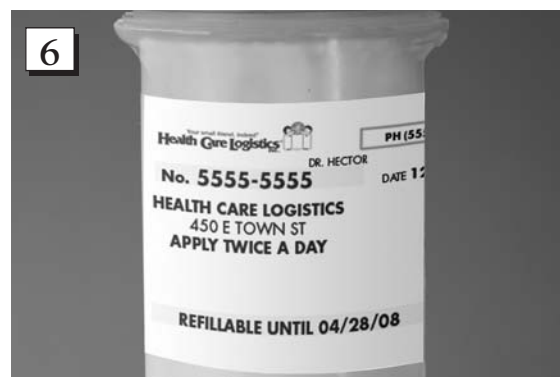

Step 6. Place prescription label on the side opposite of "Refill" indicator and graduation marks.





Step 7. For customer use, remove outer cap.



Step 8. Turn the base clockwise two clicks to dispense one dose.



Step 9. Apply cream by rubbing applicator on the affected area of skin.



Note: When empty, color indicator will be at the "Refill" line.

