



www.steri-tamp.com

INSTRUCTIONS FOR VIALS AND BOTTLES

Items: ST1102, ST900901 & ST2103

For Single-Use


Description: Steri-Tamp® provides assistance in monitoring whether injectable products have already been used by indicating “OPENED” when the metalized layer is removed.

Intended Purpose: The Steri-Tamp® purpose is to provide means to identify if a previously opened vial or IV bottle has been tampered with.

Precautions: Contents have been GAMMA sterilized and intended for single use. The target area remains sterile until removed from liner. Allied Pharmacy Products does not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

Warning: Steri-Tamp® is to be used by a trained medical professional. Steri-Tamp® cannot be a substitute for good hospital practice and procedures.

Step 1: Remove Steri-Tamp® from liner.

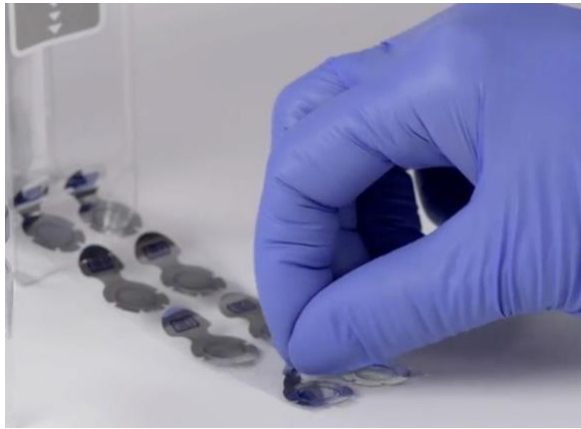

Step 2: Use thumb and index finger to position registration mark over center opening on target area.



Step 3: Press around the seal with finger to firmly affix to top.



Step 4: Upon removal, the top layer will separate from adhesive ring.

