



# ASSEMBLY INSTRUCTIONS FOR

## Economy PalWOW (#5640)

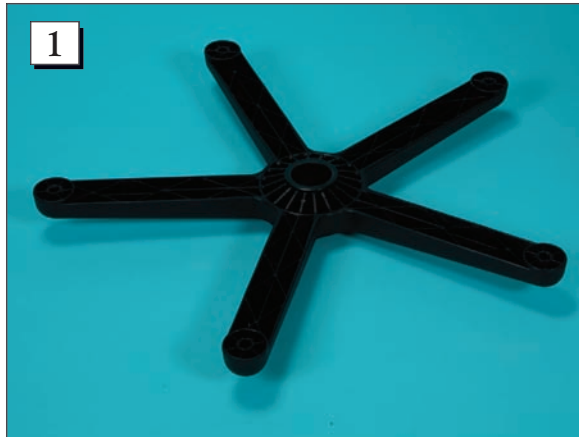

**Step 1.** Turn your base over.

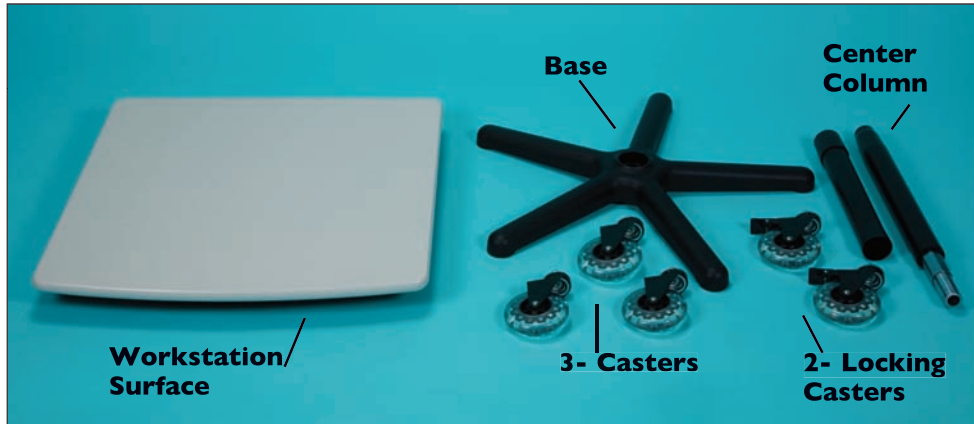

**Step 2.** Push a single caster into a leg of the base



**Step 3.** Repeat on the remaining legs making sure not to have the two locking caster next to each other.

Call Free 1.800.848.1633 or 1.888.HCL-INTL Fax Free 1.800.447.2923  
Website: [www.HealthCareLogistics.com](http://www.HealthCareLogistics.com) • Email: [hcl@HealthCareLogistics.com](mailto:hcl@HealthCareLogistics.com)

# ASSEMBLY INSTRUCTIONS FOR

## Economy PalWOW (#5640)



**Step 4.** Place the hollow center column piece in the base.



**Step 5.** Tap into place with a rubber mallet.



**Step 6.** Gather the workstation surface and place it on the floor top side down.




**Step 7.** Place the other column piece with the hydraulic lift going into the bottom of the workstation surface.


Call Free 1.800.848.1633 or 1.888.HCL-INTL Fax Free 1.800.447.2923  
Website: [www.HealthCareLogistics.com](http://www.HealthCareLogistics.com) • Email: [hcl@HealthCareLogistics.com](mailto:hcl@HealthCareLogistics.com)

# ASSEMBLY INSTRUCTIONS FOR

## Economy PalWOW (#5640)

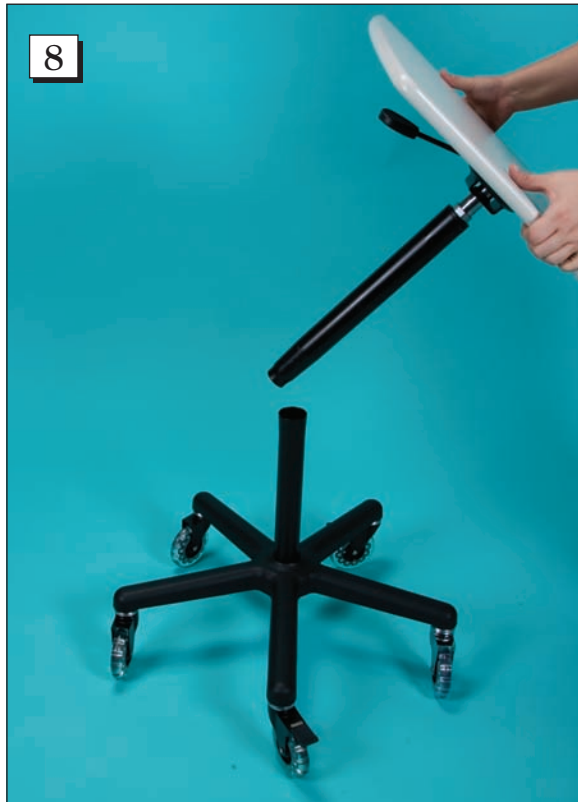

**Step 8.** Take the workstation surface and place it into the base.

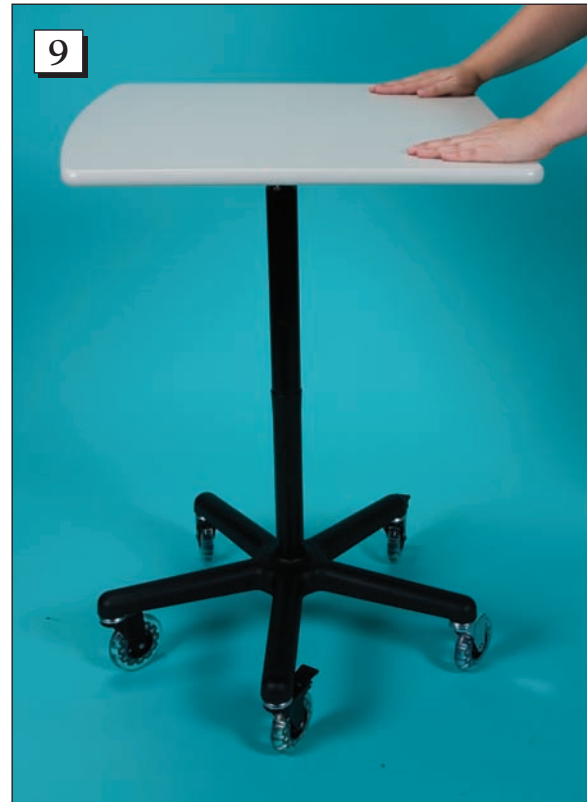

**Step 9.** Press firmly to ensure a good fit.



**Step 10.** Adjust to desired height.



Call Free 1.800.848.1633 or 1.888.HCL-INTL Fax Free 1.800.447.2923  
Website: [www.HealthCareLogistics.com](http://www.HealthCareLogistics.com) • Email: [hcl@HealthCareLogistics.com](mailto:hcl@HealthCareLogistics.com)