

INSTRUCTIONS FOR The Pod IV Lock Boxes (#19884 & 19886)



Step 1. Open the pod and loosen the locking wheel on the inside of the box, and turn the mounting knob to make room for the pole.



Step 2. Mount the pod to the IV pole and tighten the mounting knob.



Step 3. Tighten the locking wheel on the inside of the pod.



Health Care Logistics will not be liable if product is used outside of scope of the instructions.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923
Web: GoHCL.com • Email: hcl@GoHCL.com