

ASSEMBLY INSTRUCTIONS FOR

Tamper Evident Lure Lock Syringe Cap (HCL# 8003-04)



Step 1. Peel back the wrapper on the 8003-04.



Step 2. Remove one of the syringe caps.



Step 3. Place it on the end of a syringe.



Step 4. Twist the cap on until it is tight.



© Health Care Logistics, Inc. 2007 • Printed in the U.S.A.

Call Free 1 800 848 1633 or 1 888 HCL-INTL • Fax Free 1 800 447 2923
International Call 614 477 1686 • International Fax 614 477 2933 Web Site: www.hcl.intl.com • E-mail: HCL@hcl-intl.com



Step 5. When you are ready to use the syringe pull the clear cap until it releases.



Step 6. Screw off the red cap and attach your needle. Now the syringe is ready to be used.



The clear cap will not be able to be placed back on the syringe once it has been removed. If the clear cap is not on the syringe, this will indicate that the syringe has been tampered with.



© Health Care Logistics, Inc. 2007 • Printed in the U.S.A.

Call Free 1 800 848 1633 or 1 888 HCL-INTL • Fax Free 1 800 447 2923
International Call 614 477 1686 • International Fax 614 477 2933 Web Site: www.hcl.intl.com • E-mail: HCL@hcl-intl.com