

Saljet™ - Single Dose Sterile Saline Topical Solution Instructions # 13090



1. Tear a single container from a strip of four.



2. Hold the body of the container firmly, but without squeezing, and grasp the tab between your thumb and fingers.



3. Twist the tab on top of the container.



4. After twisting the tab, pull on the tab to remove it from the top of the container.



5. The container may now be positioned at any angle required to access the area to be moistened or cleaned. Apply firm pressure to the container to obtain the desired flow rate.



Saljet™ is a registered trademark of Winchester Laboratories LLC.

Sheet#13090

Rev.050801

Call Free 1 800 848 1633 or 888 HCL INTL® (425 4685) • Fax Free 1 800 447 2923

International Call 740 477 1686 • International Fax 740 477 2923 • Web site <http://www.hcl-intl.com> • E-mail: HCL@hcl-intl.com