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For CHEMOSAFETY Waste Disposal Containers and Sharps Disposal Containers with Sealing Gaskets and Hinged Lid Openings.

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English

- The use and contents of this container may dictate earlier disposal, but in no event should the container be more than three quarters full. The container should be changed regardless of contents when objects can no longer be dropped freely, without resistance, into the container.
- For more guidelines and information on how to use the sharps disposal system, in the U.S. call 1-800-962-9888 for a Sharps Safety Kit.

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© Health Care Logistics, Inc. 2006 • Printed in the U.S.A.

*Trademark of Sherwood Services AG

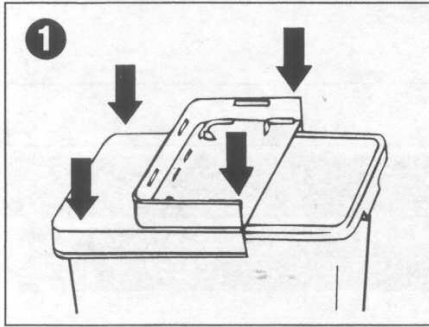
TYCO HEALTHCARE GROUP LP • MANSFIELD, MA 02048 • MADE IN U.S.A. • © 2002 TYCO HEALTHCARE GROUP LP
ALL RIGHTS RESERVED • 020529 • PRODUCT INFO IN U.S. 1-800-962-9888 • www.tycohealthcare.com

R9918G

Sheet #9607-01
Rev. 042806

Web Site: www.HealthCareLogistics.com • E-mail: hcl@HealthCareLogistics.com

INSTRUCTIONS FOR #9607-01 8 GALLON CHEMO SAFETY CONTAINER


English

1. Please check that the container has suffered no shipping damage before placing into use. Snap lid on container with hinged lid open. Listen for clicks.
2. Hinged lid may be closed temporarily when not in use by lightly closing door.

TO DISPOSE OF WASTE:

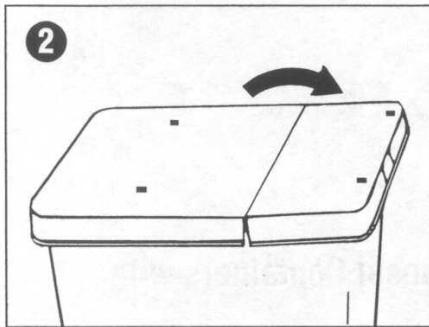3. TO USE HINGED LID – Use one hand to open hinged lid, drop waste in container and temporarily close hinged lid. **DROP ONE SHARP IN AT A TIME.**
4. FOR FINAL DISPOSAL – Lock door by pressing down securely on edges of lid. Listen for clicks. **FOR INCINERATION ONLY.**

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