
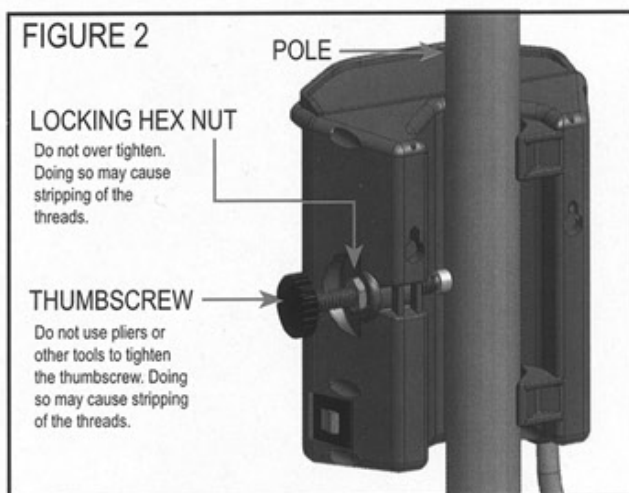


ASSEMBLY INSTRUCTIONS FOR

Powermate® (#8400)

Pole and Cart Mounting Instructions

1. Orient the unit so the splash guard is at the top and the power cord is at the bottom as shown in Figure 1.
2. Slide the unit onto the pole so that the pole rests opposite the thumbscrew as shown in Figure 2.
3. Turn the thumbscrew clockwise until the unit will not rotate on the pole under light pressure. NOTE: Do not use pliers or other tools to tighten the thumbscrew. Doing so may cause stripping of the threads.
4. Apply a wrench or pliers to the locking hex nut (shown in Figure 2) and turn clockwise until tight. NOTE: Do not over tighten. Doing so may cause stripping of the threads.

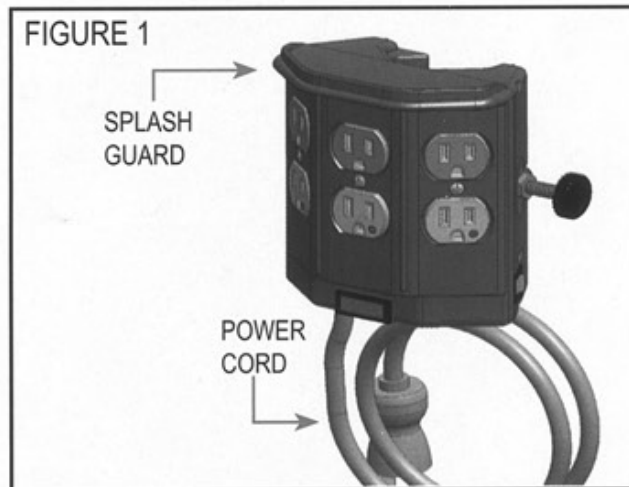

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923

Web: www.HealthCareLogistics.com • Email: hcl@HealthCareLogistics.com