

INSTRUCTIONS FOR HCL® SalJet® Sterile Topical Saline Solution (#21094 & 21094-31)



Step 1. Tear a single container from a strip of four.



Step 2. Hold the body of the container firmly, but without squeezing, and grasp the tab between your thumb and fingers.



Step 3. Twist the tab on top of the container.



Step 4. After twisting the tab, pull on the tab to remove it from the top of the container.



Step 5. The container may now be positioned at any angle required to access the area to be moistened or cleaned. Apply firm pressure to the container to obtain the desired flow rate.

Call Free: 1.800.848.1633 • Fax Free: 1.800.447.2923
GoHCL.com • hcl@GoHCL.com