



INSTRUCTIONS FOR TABLET CRUSHER (HCL #7305 and #7355)




1. Place the Individual Crushing Cup into the Tablet Crusher.

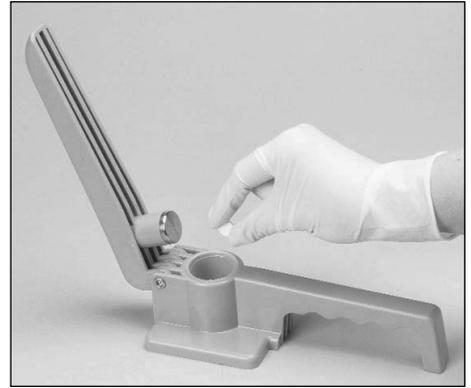


2. Place the tablet into the Crushing Cup.

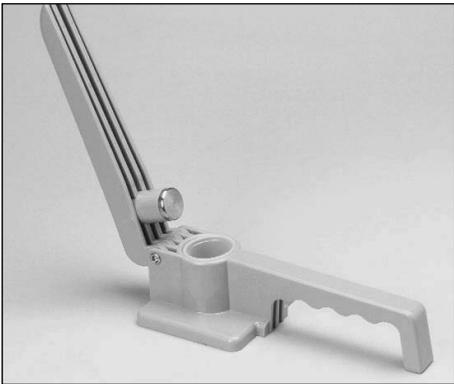


3. Cover the tablet with a second cup.

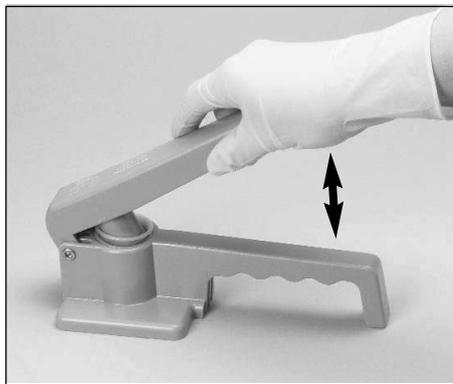

4. Use an up & down tapping motion to crush the tablet.

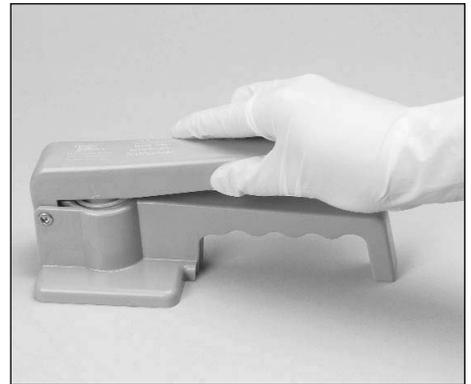

5. Excessive force is not necessary. The tapping action does the crushing.

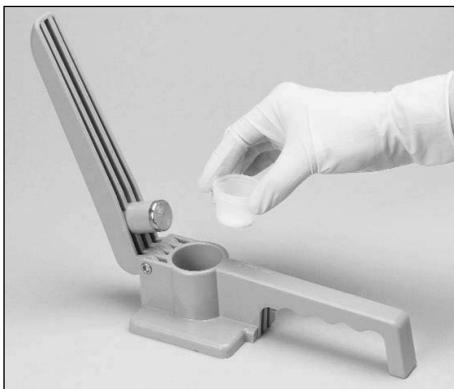

6. Remove the cups from the Crusher.



7. Administer the medication. Dispose of the cups, and you're ready to crush the next patient's medication. No clean-up is required.

Don't crush enteric coated tablets.

Distributed by
Health Care Logistics, Inc.
Manufactured by Ocelco, Inc.

rev. date 5-22-08
Sheet #17355