

INSTRUCTIONS FOR

Adhesive Hooks for Disposable Isolation Station (#17882)

**IMPORTANT: FOLLOW instructions carefully!
SAVE INSTRUCTIONS for clean removal in the future!**

To Mount Hook:



1. Clean Surface

Use isopropyl rubbing alcohol, wipe lightly let dry. DO NOT use household cleaners.

2. Apply Strip

Remove red Command™ Adhesive liner. Align square end with raised line on back of hook. Press firmly.


3. Press Firmly!

Remove black "wall side" liner. Position hook and press entire hook firmly for 30 Seconds.

4. Wait 1 hour before use.

CAUTION:
DO NOT use with delicate or paper wallpaper. May not adhere well to some vinyl wallpapers. DO NOT use for antiques, heirlooms or other valuable or irreplaceable items. DO NOT hang objects over beds.

To Remove Hook and Base:

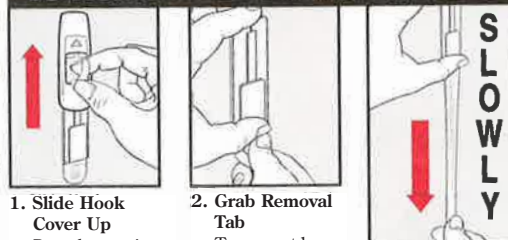

1. Slide Hook Cover Up

Reveal mounting base and remove tab.

2. Grab Removal Tab

To prevent base from snapping your fingers, hold base as shown. DO NOT press base against wall. Never pull the strip out at an angle. Strip could break or damage the surface.

3. SLOWLY
Slide the tab straight down against the wall stretching the strip at least 15" (40cm) until mounting base releases from the wall.

NOTE:

Apply to surfaces above 50°F (10°C). Adhesive could soften and lose adhesion above 105°F (40°C).