


INSTRUCTIONS FOR HCL Pump Cover for MedFusion (#20119, 20121 & 20123)



Step 1. Mount the cover to your IV pole.

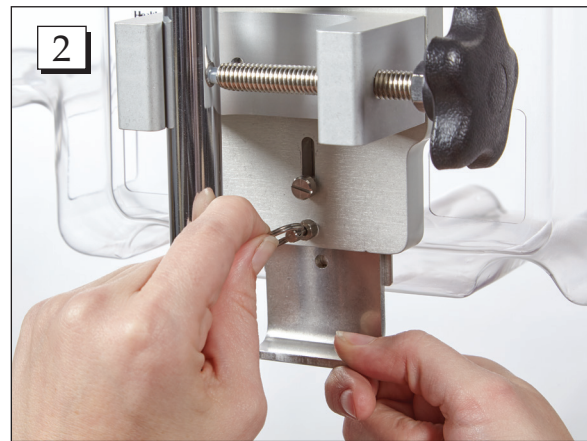

Step 2. Pull the release ring out and then pull the locking plate down to unlock the unit.




Step 3. Tilt the pump slightly towards you when inserting into the locking mount.



Step 4. Roll the pump into place. Note: When properly seated the pump cannot be lifted out of the locking mount.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923
Web: GoHCL.com • Email: hcl@GoHCL.com

INSTRUCTIONS FOR HCL Pump Cover for MedFusion (#20119, 20121 & 20123)

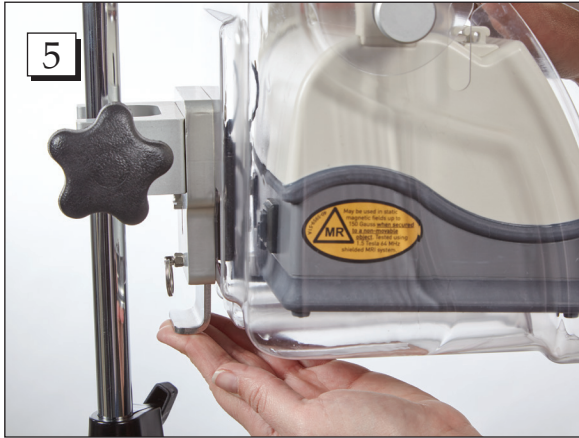

Step 5. Push the pump back against the mount and then push the locking plate up to lock the pump into place.



Step 6. To close/ release the lid pull the white tab on the inside upper left corner.

Removing the Pump from HCL Pump Cover for MedFusion (#20119-20124)



Step 7. To remove the pump, pull the release ring out and then pull the locking plate down to unlock the unit.



Step 8. Roll the pump out in a downward tilt while pressing lightly down on the bottom of the pump.

Health Care Logistics will not be liable if product is used outside of scope of the instructions.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923
Web: GoHCL.com • Email: hcl@GoHCL.com