

# INSTRUCTIONS FOR Personal Tablet Crusher (#7335, 7338)



**Step 1.** Unscrew crusher lid and remove. Then place tablet into the base of the Personal Tablet Crusher.



**Step 2.** Replace the crusher lid on the crusher base turning clockwise until the lid pressure crushes the tablet. Rotate the lid back and forth to further crush the tablet(s). Dispense the crushed medication or continue with step 3 to dilute the medication.



**Step 3.** Add water or juice to the base of the crusher and snap the cap from the storage compartment onto the base.



**Step 4.** Shake the crusher to mix the water or juice with the medication.



**Step 5.** Drink directly from the crusher or draw diluted medication into an oral syringe to be administered.

**Call Free: 1.800.848.1633 • Fax Free: 1.800.447.2923**  
Web: [GoHCL.com](http://GoHCL.com) • Email: [hcl@GoHCL.com](mailto:hcl@GoHCL.com)