

# Bag Spikes with SmartSite®



## Bag Spike with SmartSite

Sterile SmartSite needle-free bag access spike.

## Specifications

- This device does not contain DEHP in the fluid path.
- Individually packaged.



### Contraindications:

None when used as described in product insert.



- Single use only. Do not resterilize.
- Fluid path is STERILE and NONPYROGENIC. Discard if packaging is not intact or protector caps are missing or loose.



## Bag Spike with Check Valve and SmartSite

Sterile SmartSite needle-free bag access spike with check valve.



Latex-Free

## Instructions for use

Use Aseptic Technique

*Note: The valve port is two-way and luer-locking.*

- 1** Remove protective cap on spike.
- 2** Insert spike into fluid container. Suspend fluid container.
- 3** Swab valve with preferred antiseptic prior to accessing valve port.
- 4** Attach syringe to valve port.
- 5** Withdraw desired amount of fluid. Detach syringe.
- 6** Repeat steps 2 through 4 for subsequent fluid withdrawals.

*NOTE: The check valve prevents medication additions to the solution container.*



			Order No. REF.
Bag Spike with SmartSite	1	20	H938 93 5
Bag Spike with SmartSite & Check Valve	1	20	H938 94 5

# Vial Adapter with SmartSite®



Sterile SmartSite needle-free vial access device for multiple withdrawals or additions of fluids.

## Specifications

- Fluid path contains no DEHP, DOP plasticized PVC.
- Individually packaged.



### Contraindications:

None when used as described in product insert.



- Single use only. Do not resterilize.
- Fluid path is STERILE and NONPYROGENIC. Discard if packaging is not intact.



Latex-Free

## Instructions for use

Use Aseptic Technique

*Note: The valve port is two-way and luer-locking.*

- 1 Swab septum with preferred antiseptic prior to accessing valve port.
- 2 Attach access device to vial.
- 3 Connect syringe to needle-free valve port.
- 4 Withdraw or add desired fluid volume.



			Order No. REF.
Vial Adapter with SmartSite	1	20	H938 92 5



### Corporate Headquarters - USA

**Baxa Corporation**  
 13760 E. Arapahoe Rd.  
 Englewood, CO 80112-3903  
 tel: 303-690-4204  
 fax: 303-690-4804  
 toll free: **800-567-BAXA (2292)**  
 E-mail: [baxamail@baxa.com](mailto:baxamail@baxa.com)  
[www.baxa.com](http://www.baxa.com)

### Registered Agent - United Kingdom

**Baxa Limited**  
 Glendale Park  
 Fernbank Road  
 North Ascot  
 Berkshire SL5 8JB  
 tel: +(44) 1344 890916  
 fax: +(44) 1344 890917  
 E-mail: [enquiries@baxa.co.uk](mailto:enquiries@baxa.co.uk)

### Canada

**Baxa Canada**  
 210 Cochrane Dr. Unit #2  
 Markham, Ontario  
 L3R 8E6  
 tel: 905-946-0319  
 fax: 905-946-1710  
 toll free: 800-357-BAXA (2292)  
 E-mail: [baxamail@baxa.ca](mailto:baxamail@baxa.ca)

To find a local Baxa agent, visit our website at [www.baxa.com](http://www.baxa.com) or call our offices directly.