


# ASSEMBLY INSTRUCTIONS FOR

## Easy Empty Crusher Bags (#12679)

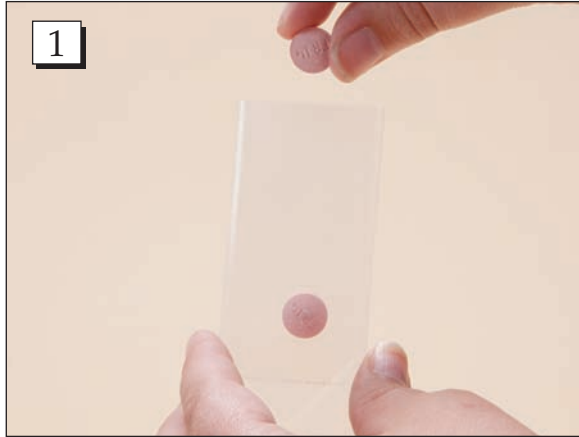

**Step 1.** Remove a crusher bag from the storage box located in the crusher's bag storage area. Place tablet(s) in the crusher bag



**Step 2.** Remove a crusher bag from the storage box located in the crusher's bag storage area. Place tablet(s) in the crusher bag



**Step 3.** Press the handle gently down to crush the tablets. The handle will need to be pressed down 2 or 3 times while the tablet crusher bag is in position. Do not tap. Only light pressure is required to crush tablets.



**Step 4.** Crushed tablets are ready to be administered from the crusher bag.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923  
Web: GoHCL.com • Email: hcl@GoHCL.com

# ASSEMBLY INSTRUCTIONS FOR

## Easy Empty Crusher Bags (#12679)



**Step 5.** Position the bag over juice, applesauce, pudding, or cup. Tear bottom of bag at perforation.



**Step 6.** Pull to remove the entire perforated strip.



**Step 7.** Squeeze the outside of the bag and the unique funnel shaped bag bottom allows the crushed medication to easily pour out.

**Note:** The unique perforation design allows medication to travel less distance while emptying, ensuring a more accurate dose delivery.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923  
Web: GoHCL.com • Email: hcl@GoHCL.com