

ASSEMBLY INSTRUCTIONS FOR Handle Kit for IV Solution Cart (#5266)



Note: You will need a small rubber mallet to secure the handle in place.



Step 1. Gather one sets of plastic sleeves. Choose the height you want to set the handle at. Place the first clip on the pole.



Step 2. Place the second half of the plastic sleeve on the pole.



© Health Care Logistics, Inc. 2008 • Printed in the U.S.A.

Call Free 1.800.848.1633 or 1.888.HCL.INTL • Fax Free 1.800.447.2923
Web Site: www.HealthCareLogistics.com • E-mail: hcl@HealthCareLogistics.com



Step 3. Squeeze the two parts together until they click together.



Step 4. Repeat steps 1-3 on the other side.



Step 5. Slide the handle over the pole ends



Step 6. Tap into place with a rubber mallet.

