

Saljet™

Single dose sterile saline topical solution.

4 units of sodium chloride 30ml (0.9% w/v).

Indications: For use in moistening and lubricating absorbent wound dressings for traumatic wounds, cuts, bruises and minor burns prior to removal from the wound area.

Caution: Do not use unless the product is clear. If there is any evidence of leakage prior to breaking the seal, discard. Discard any unused portion. Does not contain antimicrobial or other ingredients. For external use only. Not for injection.

Rx only.


Directions for use: Use undiluted. Twist off the top and apply topically to the affected area with a firm pressure on the Saljet vial providing a steady stream of the solution. To obtain drop by drop dispensing, invert the vial and squeeze gently.

Lot No: AF 033


Expiration Date: 2002 06

Manufactured for Winchester Laboratories LLC, St. Charles, Illinois 60174.
Made in Germany


Directions For Use

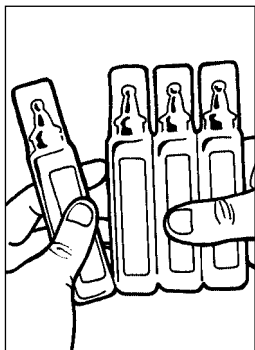


1. Tear a single container from a strip of four.

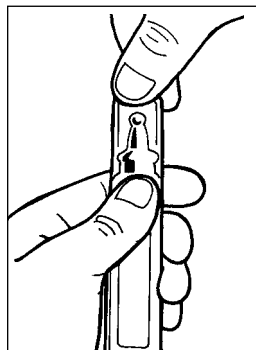


2. Hold the body of the container firmly but without squeezing, then grasp the tab between your thumb and forefingers.

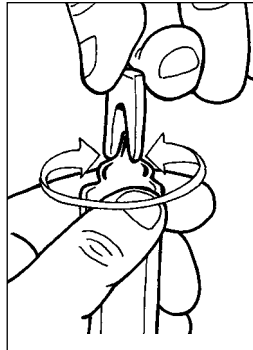

3. Twist the tab on top of the container.

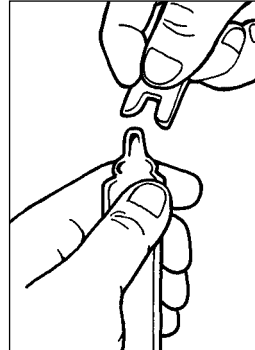

4. After twisting, pull on the tab to remove it from the top of the container.

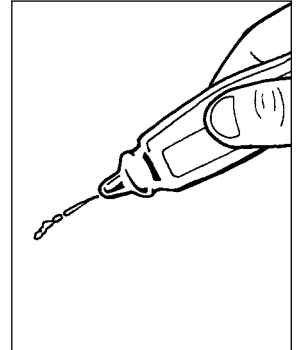

5. The container may now be positioned at any angle required to access the area to be moistened or cleaned. Apply firm pressure to the container to obtain the desired flow rate.